




# enchroma®

.....  
**READ THIS FIRST**  
.....



## GO OUTSIDE

Outdoor glasses work best outside. Remember to capture the moment with your friends.

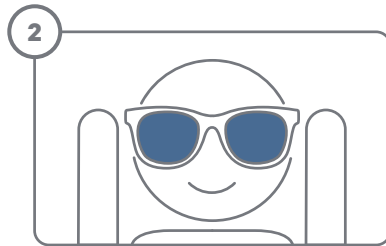


## PUT THEM ON

The big moment is here! Put on your new Enchroma glasses and take a look around.



## KEEP THEM ON

No peeking! Keep your glasses on for 10-15 minutes to give your eyes time to adjust.

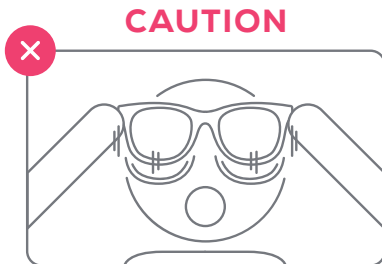

Avoid “flipping” the glasses so your eyes can adjust to the new lenses. For best results, wear them for 10+ hours over the course of 1-2 weeks.



Wear indoor glasses in well-lit interiors, at the movies, or in front of your TV or computer screen. They work best when there is a bright light source.

## CARE AND HANDLING



Avoid touching the lenses



DO NOT USE abrasive cleaners



Store glasses in a cool, dry area

Avoid touching the lenses with your fingers as that may build up oil and residue.

Wash lenses with mild soap and warm water. Rinse clean and remove excess water with a soft cleaning cloth. DO NOT use paper towels, ammonia, alkaline cleaners, window cleaners, solvents or abrasive cleaners as it can scratch or damage the lenses.

Store Enchroma glasses inside a protective case when not in use. Store in a cool, dry area away from extreme temperatures as it can damage the lenses. DO NOT store the glasses inside hot vehicles.

Questions? Visit [enchroma.com/contact-us](http://enchroma.com/contact-us)

Enchroma glasses are designed for use with deuteranomalous or protanomalous color vision deficiency (more commonly known as red-green color blindness).

The effects of the product may vary depending on one's type and severity of color vision deficiency.

Some people will experience a profound effect immediately, while it might take others longer. In some cases, it may take up to two weeks to see results. The effects of the glasses tend to become stronger the longer they are worn.

When wearing the glasses for the first time, allow your eyes to comfortably adjust to the glasses for about 10-15 minutes. Refrain from removing the glasses during this period, as it may reduce their effectiveness.\*

\*Results may vary

During the initial adaptation period, objects may become visually distracting. Always maintain awareness of your surroundings.

NOTE: Enchroma glasses are designed as an optical aid for people who have red-green color blindness. The glasses are not intended to diagnose, treat or cure any condition related to color vision deficiency.

Enchroma glasses are not designed to pass color blindness screening tests.

Enchroma glasses are not certified for use in occupations requiring color recognition, such as fields in public safety and/or transportation.

WARNING: Enchroma lenses are impact resistant and have a scratch resistant coating. LENSES ARE NOT SHATTER-PROOF. DO NOT USE as safety glasses. DO NOT USE in any activity that may harm your face and eyes. DISCONTINUE USE IMMEDIATELY IF LENS IS CHIPPED OR CRACKED.

ENCHROMA LENSES MAY NOT COMPLY WITH AMERICAN NATIONAL STANDARDS INSTITUTE Z80.3-2010 SECTION 4.6.6.3. NOT FOR USE WHILE DRIVING.

### Follow Us



[facebook.com/enchroma](https://facebook.com/enchroma)



[twitter.com/enchroma](https://twitter.com/enchroma)



[youtube.com/enchromainc](https://youtube.com/enchromainc)



[instagram.com/enchroma](https://instagram.com/enchroma)

### Warranty

Enchroma provides customers with a warranty of one (1) year from the date of purchase. For more information please visit:

[enchroma.com/warranty](http://enchroma.com/warranty)

Enchroma Inc.  
2629 Seventh Street  
Berkeley, CA 94710

Phone: +1 (510) 497-0048  
Email: [info@enchroma.com](mailto:info@enchroma.com)  
Web: [enchroma.com](http://enchroma.com)

Enchroma and See the Difference are trademarks of Enchroma, Inc.

## JOIN US ONLINE

We love hearing your stories and seeing your videos!  
Find us online to see and share the Enchroma experience.

#enchroma

